
EASY BREATH™

EASY BREATH™ is a disposable device for heat and moisture exchange (HME or artificial nose) to be used on tracheostomized spontaneously breathing patients, with a port for administration of oxygen if necessary.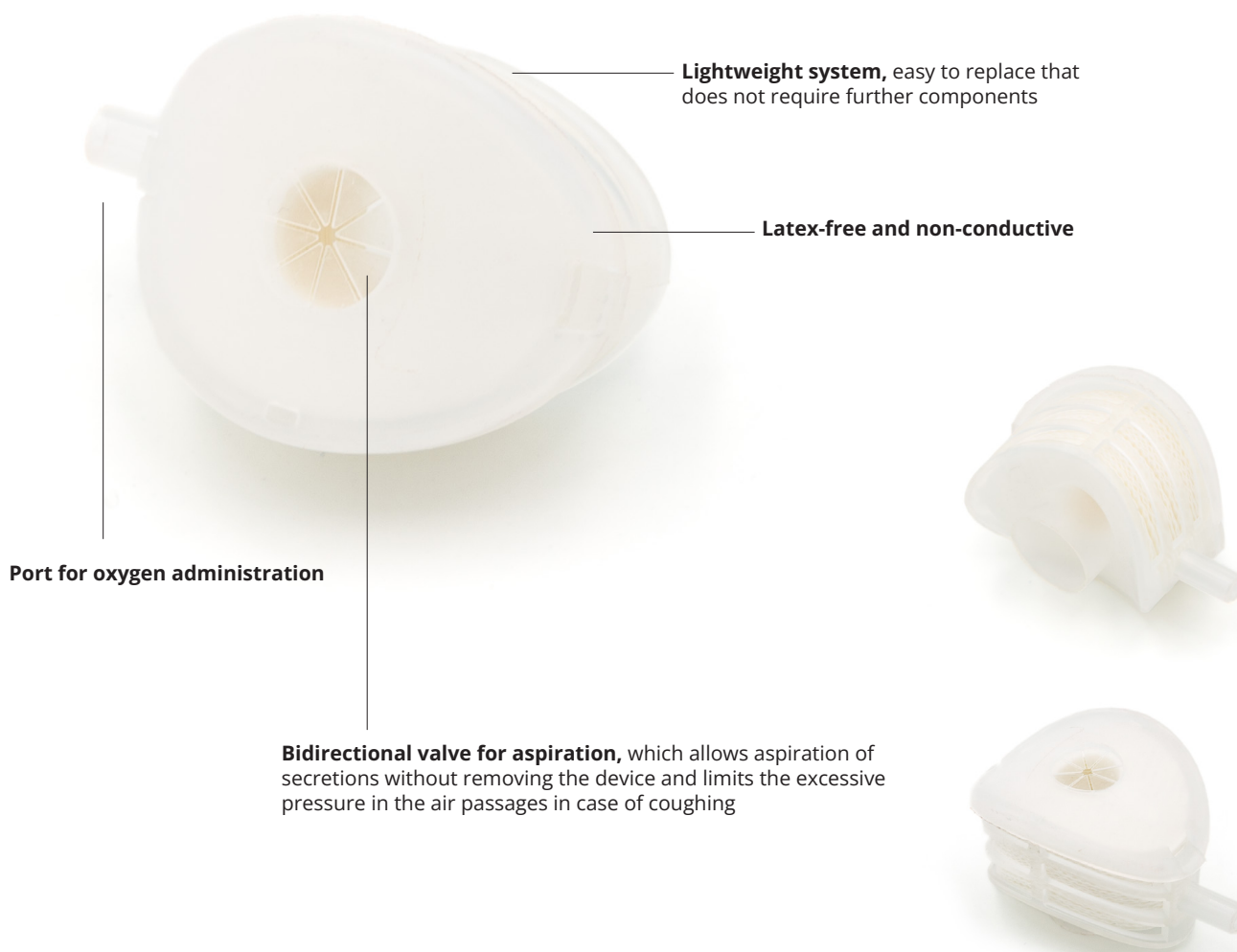

Port for oxygen administration

Lightweight system, easy to replace that does not require further components

Latex-free and non-conductive

Bidirectional valve for aspiration, which allows aspiration of secretions without removing the device and limits the excessive pressure in the air passages in case of coughing

How to use:

- + Before use, make sure the packaging is intact. Then, **take the device out and connect it firmly to the tracheostomy tube.**
- + If extra oxygen is needed, **simply connect the oxygen tube to the side port of the device.**
- + Thanks to the special valve, you can carry out suctioning or inspections without removing the device.
- + **Check regularly for secretions that may block the filter.** If you notice a lot of buildup, replace the device.
- + For safety and hygiene: **use only on one patient**, for up to 24 hours, then dispose of it following local rules.

TECHNICAL FEATURES	
FILTER VOLUME	20 ml
WEIGHT	7,8 gr.
CURRENT VOLUME	Not to be used with volumes < 100 ml
EXCHANGE SURFACE	500 cm²
FLOW RESISTANCE	0,28 cm/H2O – 60 l x min. 0,20 cm/H2O – 30 l x min.
HUMIDIFICATION EFFICIENCY	29,6 mg /l – Vc = 500 ml
CONNECTIONS	15 F
MAXIMUM DURATION OF USE	24h
PACKAGING	In individual bags in boxes of 100 pcs.
SYSTEM OF USE	Single-use, single-patient device

ORDER GUIDE - EASY BREATH™

CODES:

- BK008BN (sterile)
- BK008XBN (produced in a controlled environment)

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Società soggetta ad attività di direzione e coordinamento da parte della società Bpunto3 S.r.l.